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FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION, CONTACT CSI AT 1-800-882-0116.

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[www.homecare.moen.com](http://www.homecare.moen.com)

INNOVATIVE DESIGNS  
for independent living™

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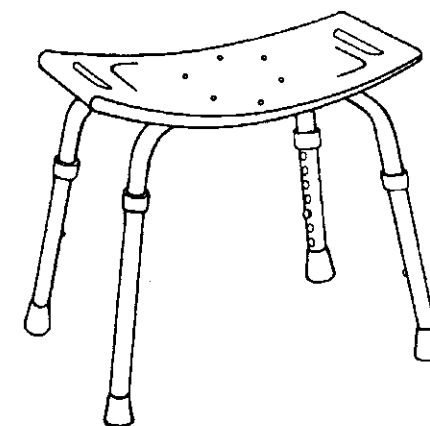
**Dimensions:** Adjustable height from 14" to 21"  
20" wide x 12" deep x 16" high

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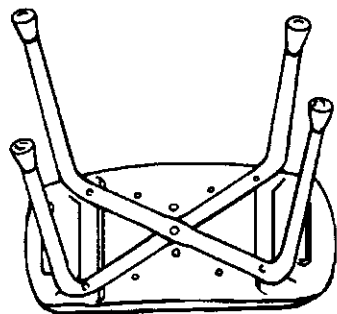
**Tools Required:**

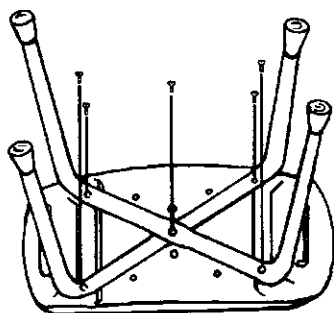
- Standard size Phillips head screwdriver

**Assembly Instructions****Step 1: Position Legs**

Place leg "1" into groove "1" of the plastic seat panel so that the number "1" marks are next to each other. Then place leg "2" into groove "2", over leg "1" so that the number "2" marks are next to each other.

**Step 2: Attach Legs**

Insert a 1 1/2" screw through a washer and the center hole of the legs and seat. Tighten several turns. Insert remaining four screws through washers into the four leg holes. Tighten all five screws fully. Test seat. Retighten as needed.

**Step 3: Adjust Leg Height (DN7025)**

Push in button on leg, slide leg to desired height and allow button to pop out and lock leg in place. Repeat for each leg, making sure all four are adjusted to the same height.

**WARNING**

This product has a maximum user weight of 250lbs. Do not jump or stand on this seat. Contact CSI immediately at 1-800-882-0116 if any components are damaged or missing. Making substitutions or modifying this product in any way will void your warranty. CSI assumes no liability for any damage or injury caused by improper assembly, installation or use of this product.

**Cleaning instructions:**

Wipe clean with a soft, damp cloth. Detergents and cleansers may harm the finish of this product.

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Home Care is a registered trademark of Home Care Tim-BR-MART, licensed to Moen Incorporated. / Home Care es una marca registrada de Home Care Tim-BR-MART, cuyo uso fue autorizado a Moen Incorporated. / Home Care est une marque déposée de Home Care Tim-BR-Mart et Moen incorporée en est un utilisateur autorisé.